

Quick Start Guide

Wired Nurse-call Floorgard

Applies to product codes:

R01-0011A-I | R01-0203A-I

System Contents	
Sensor(s)	RG58 Wired Deluxe 90 Floorgard RG15 Wired Standard Floorgard
Lead(s)	RG7 3M breakaway lead RG8 8M breakaway lead
Adapter	Custom hard-wired nurse-call connector RG4 Jack Adapter RG15 Mono/Stereo switched plug with spare call socket

Sensor Placement

Floorgard: Place mat at or near to the area to be monitored such as a doorway or side of the bed.

General Operation

When a sensor mat is triggered, the nurse-call system is alarmed. Can only be reset at the nurse-call point.



Quick Start Guide

Wired Nurse-call Floorgard

Follow the steps below to set up your equipment:

- 1.** Attach the supplied nurse-call cable to the floor sensor mat via the breakaway point.
- 2.** Place the floor mat in the desired location according to the level of monitoring that is sought. Route the cable in a safe and convenient manner which will not cause a trip or entanglement hazard.
- 3.** Ensure that the nurse-call plug matches the socket of your nurse-call system by performing a visual check. If they do not match or you are unsure, please contact your area sales representative for advice.
- 4.** Check that you have positioned your sensor pad correctly and connect the cable.
- 5.** Confirm the mono/stereo switch and the on/off switches are in the correct position.
MONO/STEREO - refer to your nurse-call system configuration
ON/OFF - only turn ON when a nurse-call button is inserted to the call cord socket.
- 6.** To test, activate the sensor to confirm that it is alarming, triggering your nurse-call and resetting correctly.

When you are satisfied with the points above please complete the daily checks sheet provided before putting the system into use.

**Ramblegard Technical Support Helpline:
028 9448 3979**

